# ORBIS

# ORBIS

# ResinoCal LC

**EN** Bioactive light cure pulp capping material with MTA-fillers

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## ORBIS ResinoCal LC Instructions for Use

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**EN** **PRODUCT DESCRIPTION**  
**ORBIS ResinoCal LC** is a bioactive light cure resin reinforced pulp capping material with MTA-fillers. It has been designed to perform as a barrier and to protect the pulp. Thanks to its thixotropic behavior, the material can be applied very precisely even in deep cavity preparations using the supplied Orbis ResinoCal LC tip. As it is light-curable, **ORBIS ResinoCal LC** ensures controlled setting during placement.

**INDICATIONS/INTENDED USE**

**1. Direct pulp capping for any pulpal exposures, including:**

- carious pulp exposure
- mechanical pulp exposure
- pulp exposures due to trauma

**2. Indirect pulp capping agent in deep preparations:**

- under amalgam restorations
- under Class I and Class II composite restorations
- under cements
- as an alternative to calcium hydroxide

**Performance features**

The performance features of the product meet the requirements of the intended use.

**Contraindications**

In singular cases, the material may cause a sensitizing reaction in patients with a hypersensitivity to any of the ingredients. In these cases, the material should not be used.

**Patient target group**

Persons who are treated during a dental procedure.

**Intended users**

This medical device should only be used by a professionally trained dental practitioner.

**Precautions**

- Because of its high pH value **ORBIS ResinoCal LC** must not be used in direct intraoral environment. Do not place on enamel, cavity margins or leave exposed to oral environment.
- The success of pulp capping procedures may be affected by preparation contamination. Use of a rubber dam or adequate isolation is recommended.
- Pulpal hemorrhage must be controlled prior to the application of **ORBIS ResinoCal LC**. If hemorrhage cannot be controlled, consideration should be given for the initiation of endodontic therapy before direct pulp capping procedures.

**Application**

Screw off the cap of the syringe and substitute it by a Orbis ResinoCal -Tip. **Store the screw cap!**

**1. Direct pulp capping for any pulpal exposures**

**1.1. Cavity Preparation**

When opening the pulp, first stop any bleedin e remove excess moisture with a sterile cotton pellet. Leave the surface visibly moist.

**1.2. Application**

Apply **ORBIS ResinoCal LC** punctiform and small-area (with a layer thickness ≤ 1 mm) to the opened pulp. Cover all exposed areas of the opened pulp with **ORBIS ResinoCal LC** and extend optional up to 1 mm into dentine. Due to the high opacity of **ORBIS ResinoCal LC** light cure 40 seconds by using a polymerization unit (wavelength range 400-500 nm) with a light intensity of at least 1000mW/cm².

Apply a suitable light cure adhesive according to the manufacturer instructions onto the cured **ORBIS ResinoCal LC** layer and proceed with the restoration.

**Note:**

With a total-etch adhesive take care to etch only the exposed cavity areas. Then apply the bonding to all cavity areas.

**2. Indirect pulp capping agent in deep preparations**

**2.1. Cavity Preparation**

Remove all infected carious tooth structures. Leave the preparation visibly moist.

**2.2. Application**

Apply **ORBIS ResinoCal LC** small-area (with a layer thickness ≤ 1 mm) directly onto the area close to the pulp. Due to the high opacity of **ORBIS ResinoCal LC** light cure 40 seconds by using a polymerization unit (wavelength range 400-500 nm) with a light intensity of at least 1000mW/cm².

Apply a suitable light cure adhesive according to the manufacturer instructions onto the cured **ORBIS ResinoCal LC** layer and proceed with the restoration.

**Note:**

With a total-etch adhesive take care to etch only the exposed cavity areas. Then apply the bonding to all cavity areas.

After use of the syringe replace disposable **ORBIS ResinoCal -Tip** by the screw cap. **ORBIS ResinoCal -Tip are for single use only.**

**Storage**

Store at 2 – 8 °C (36 – 46 °F) in the original seal-pack. After opening of the seal-pack store the material protected from light and moisture at room temperature (19 - 25 °C / 66 - 77 °F). Use up within 6 months and before the end of expiry date. Close syringe immediately after use. Do not use after expiry date.

**Additional Notes:**

- ORBIS ResinoCal LC** contains polymerisable methacrylate monomers. Unpolymerized materials may have an irritating effect and may lead to a sensitizing reaction against methacrylates.
- Do not use with patients with allergic reactions against acrylates.
- Avoid contact with skin, mucous membrane and eyes.
- In case of contact with eyes, rinse with copious amounts of water and consult a physician immediately.
- In case of skin contact wash immediately with water.
- Do not use in combination with substances containing eugenol because eugenol inhibits the polymerization of the material. Neither store the material in proximity of eugenol containing products, nor let the material allow coming into contact with materials containing eugenol.
- Commercial medical gloves do not protect against the sensitizing effect of methacrylates.
- Keep away from children!

**Disposal**

Disposal of the product according to local authority regulations.

**Reporting obligation**

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

**Warranty**

First Scientific Dental GmbH warrants this product will be free from defects in material and manufacture. First Scientific Dental GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user’s application. If this product is defective within the warranty period, your exclusively remedy and First Scientific Dental GmbH’s sole obligation shall be repair or replacement of the First Scientific Dental GmbH product.

**Limitation of Liability**

Except where prohibited by law, First Scientific Dental GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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