EN Instructions

Before using the ORBIS RC BLUE FILE, please see the IFU as below:

Warnings

- Only skilled dentists are allowed to use. For Dental use only
- ORBIS RC BLUE FILE are sterile, single use instruments, designed for the preparation and the retreatment of no more than one tooth. Files should not be reused.
- Do not use this product except for the dental service and treatment Use it in accordance with the intended use

Contraindications and prohibitions

Do not use this product for a patient who indicates sensitization and allergic reaction. Examine the instrument during treatment for signs of wear such as untwisting, bending or damaged cutting edges. In these cases, instruments should be discarded.

Structures and principals

1. These instruments are to be used only in a clinical or hospital environment, by qualified users. Application field: shaping, cleaning and retreatment of the root canal system.

- Main materials
 - Working Cutting part: nickel-titanium alloy.
 - Shank: Copper (Nickel-Chrome-24 Gold included)

Purpose of use and effect-efficacy

This is the dental instrument with columnar or tapering working end and it possesses circular form in cross-cut section and flat end. This is used for mainly filling shape root canals.

Specifications

Under the provisions of ISO 3630 performance test, product durability for twisting and bending must exceed specified value of our standard.

Method of use

According to its intended use, choose the most appropriate ORBIS RC BLUE FILE and use it for enlarging and debriding root canal.

First create a straight line access to the root canal entrance, then create a glidepath. In most cases, the R25 will be suitable in size for the root canal treatment. Consult the pre-operative radiograph to see if the canal is likely to be considered as narrow, medium or wide. If the canal is partially or completely invisible on the radiograph: The canal is considered narrow = use the R25. If the canal is completely visible on the radiograph: Take an ISO size 30 hand instrument; insert it passively into the canal. If it reaches working length, the canal is considered large = use the R50. If an ISO size 30 hand instrument does not go passively to working length, try an ISO size 20 hand instrument. If this goes passively to length, the canal is considered medium = use the R40. If a size 20 hand instrument does not go passively to length, the canal is considered narrow = use the R25.

Passively means that the instrument goes directly to working length without any watch-winding and can be moved up-and-down without filing action.

Precautions for use

- The RC BLUE FILE designed for single use in maximum one molar.
- As with all nickel-titanium instruments, it should be examined during the treatment and discarded if signs of wear, such as untwisting, can be seen
- Irrigate abundantly and frequently due to the fast preparation.
- Place irrigant in the access cavity of the root canal. Check that motor has been set to reciprocating motions.
- Move the instrument in a slow in-and-out pecking motion. The amplitude of the in-and-out movements should not exceed 3 mm. Only very light pressure should be applied. The instrument will advance easily in the canal. 1 in-and-out movement = 1 peck. After 3 pecks, remove the instrument from the canal. Clean the debris from the flutes in the stand. Make sure the canal is free to approx. 3 mm beyond the prepared canal section with an ISO size 10 K-File. In this way, continue with the RC BLUE FILE instrument until approx. 2/3 of the working length has been reached. When using an R25; determine the working length by using an ISO size 10 K-File. When using an R40 or R50; the working length should be re-checked with an apex locator. Continue with the RC BLUE FILE until full working length has been reached.
- These instruments are to be used only in a clinical or hospital environment, by qualified users following good dental practices.
- These files must only be used in reciprocating motion with appropriate specialties motors.

Storage and duration of use

- Avoid storing at high temperature, humidity and direct sunlight. Keep liquids away. Store it at room temperature.
- Do not damage or make a pinhole to packaging materials.
- This product is subject to be improved without previous notice. Enforce first-in-first-out method for stock management.
- Do not store the instrument under a germicidal lamp to avoid deterioration.

Maintenances and inspections

If you have accidentally made a hole in the packaging or are unsure whether the ORBIS RC BLUE FILE is still sterile, sterilize this product by autoclave under established method and term before usage.

Method of sterilization:

Put this product in sterilization pack (or foil) and place it on a sterilization tray for autoclave sterilization with reference to the following terms.

Terms for sterilization:

Sterilize with steam at 134°C (273°F) / 2,1 bar / 18 min. for endodontic instruments-and filling.

Do not use high pressure steam sterilizer which heats more than 200°C including drying process.

Regarding use of medical cleaning agent, follow the instruction manual by its manufacturer strictly.

Dispose the product if damaged or decreased in performance.

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Symbol	Symbol Title	Symbol	Symbol Title
	Handle Right angle RA		Recipro motion
	Nickel titanium		Reference number ISO 13485
	Silicone		CE Certificate
	Catalogue number		Manufacturer
	Batch code		Authorized representative in the European Community
	Single use		Caution
	Consult instructions for use		Use-by date
	Sterilized by Radiation		do not use if package is damaged

CE0197 **STERILE** **R** **MD**

EC REP **SUNGO Europe B.V.**

Olympisch Stadion 24 | 1076DE Amsterdam, Netherlands

Shenzhen Perfect Medical Instruments Co., Ltd

Address: 1F, Bldg. C Plant, No.3, Jixia Zaocheng Industrial Area, Nanwan St., Longgang Dist. | Shenzhen, China
Tel: 0755-28540953 | fax: 0755-28540953 | E-mail: sales@denal-perfect.com | www.dental-perfect.com